GBR ACCESSORY FOR EXTRACORPOREAL CIRCUIT

-STERILE AND NON PYROGENIC FLUID PATH.
-OPERATE UNDER ASEPTIC CONDITIONS.
-DO NOT USE IF PACKAGE IS DAMAGED OR IF TERMINAL PROTECTION CAPS ARE NOT FULLY IN PLACE.
-USE THE DEVICE IMMEDIATELY AFTER REMOVING PROTECTION CAPS PLACED AT ITS ENDS.
-NOT MADE WITH NATURAL RUBBER LATEX.

INSTRUCTIONS FOR USE

- Follow instructions given in the manual provided by machine manufacturer. Take the line out of the pouch and connect it to the proper seats.
- Make sure that all connections are tight and proper before starting the treatment.
- Fill the system following instructions reported on the sheet provided by machine manufacturer.
- Check the line carefully and make sure that all connections are tight and that there is no leakage.
- Discard after use according to indication provided for potential contaminated product.

SPECIAL WARNING

-The line could be damaged because of improper contact with chemicals or drugs. Do not use such products unless authorized by the manufacturer.
-We disclaim all responsibility for damages caused by improper use of the product when directions for use are not followed.

CAUTION

- Fluid path sterility is guaranteed only if the packaging is not damaged and if the protective caps are perfectly placed. Use the device right after packaging opening and protective caps removal.
- Single use device, reuse may involve risk of contamination or compromised device function or structural integrity and cause illness or death.

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HAEMOTRONIC S.p.A.